

Neste generasjon FlexPen® - ferdigfylt penn med moderne insulin



**Novo Nordisk lytter til brukerne
og gjør FlexPen® enda bedre**


Enkel å lære - enkel å bruke

Noen praktiske tips før du starter

- Skru nålen rett og fast på FlexPen®
Sjekk at det kommer ut insulin
- Ved innstilling av dose gir hver enhet et tydelig klikk
- Trykk trykknappen langsomt helt inn og vent i minimum 6 sekunder før nålen dras ut
- Skru av nålen. FlexPen® skal oppbevares uten nål
- For mer informasjon, les pakningsvedlegget


Injeksjonsteknikk

Slank voksen
6 mm nål

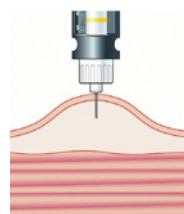

90°
mage eller lår
med hudfold

Overvektig voksen
6 eller 8 mm nål

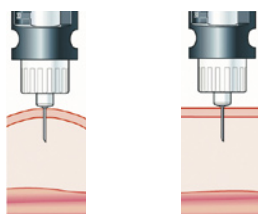

90°
lår med
hudfold

90°
mage uten
hudfold

Nye FlexPen® - nye fordeler



Lettere å trykke

– enklere å injisere^{1,2}

Tydligere fargekode

– enklere å identifisere



Bedre doseringsnøyaktighet enn
andre ferdigfylte insulinpenner³



Neste generasjon FlexPen® inneholder samme typer insulin som tidligere

- 1 Pfützner A et al. Prefilled insulin device with reduced injection force: patient perception and accuracy. *Curr Med Res Opin* 2008; 24: 2545-49
- 2 Rissler J et al. Evaluation of the injection force dynamics of a modified prefilled insulin pen. *Expert Opin Pharmacother* 2008; 9: 2217- 22
- 3 Asakura T et al. Dosing accuracy of two insulin pre-filled pens. *Curr Med Res Opin* 2008; 24: 1429-34

Novo Nordisk Scandinavia AS

Tlf. 67 17 85 00

www.novonordisk.no

www.diabetesinfo.no

Kundeservice: kundeservice-norge@novonordisk.com

